

CADI OPTIVIEW MAINTENANCE SHEET

Daily maintenance

When performing maintenance, disconnect the USB plug.

Main unit: Camera handpiece

- Use clean gauze containing ethanol for disinfection to wipe away the dirt. If the amount of ethanol for disinfection is too great when doing this, it might seep into the main unit from the connection parts, causing the product to malfunction. Also, do not disinfect by soaking or spraying.
- When the lens surface is dirty, a clear image cannot be captured. The lens must be cleaned. Gently wipe the lens surface.
- Because the quality or shape of the Camera handpiece could be altered, do not use the following disinfectants, solvents, detergents, etc.
 - Chemicals containing acetone, iodine, or phenols
 - Organic solvents such as benzine and thinner
 - Detergents such as kitchen detergent
- Also, if chemicals besides those used for daily maintenance get on the product, immediately wipe them away with ethanol for disinfection.
- Do not perform sterilization treatment using an autoclave or similar device. This could cause the product to malfunction.

Camera cable

- Wipe with a soft cloth that contains a neutral detergent.
- When performing the maintenance of the Camera cable, do not bend, tug and twist it to avoid damaging the Camera cable.

Daily check

Check that there are no chips, cracks or dirt in the Camera handpiece or Camera cable. Do not use damaged devices. Before use, check that each part is operating correctly and safely. Not doing so could cause unexpected accidents or malfunction.

Setting / storage location

Use care in selecting the setting / storage location. Not doing so could cause the product to malfunction.

- The acceptable ranges for ambient temperature, relative humidity, and atmospheric pressure are shown below.

	Ambient temperature	Relative humidity	Atmospheric pressure
Usage conditions :	5 °C to 35 °C	10 % to 80%	700 hPa tp 1060 hPa
Storage and transport conditions:	-20 °C to 60 °C	5 % to 90%	700 hPa tp 1060 hPa

- Do not expose to direct sunlight for a long period of time.
- Do not place in an unstable location.
- Do not use with a base of Camera cable bent.
- When using a product that has not been used for a long period of time, you must check that it operates normally and safely before use.